

Sheep50K



Animal Health
Animal Genetics

PO Box 5520
Dunedin, New Zealand
Ph: 0800 228 278
Fax: +64 3 4775930
pahgenetics.nz@pfizer.com
www.pfizeranimalgenetics.co.nz

SAMPLE COLLECTION FOR Sheep50K TESTING

Sheep50K is a DNA based laboratory test. This test requires a whole blood tube or tissue sample to be taken from animals to ensure adequate concentrations of DNA are received.

We require no less than 5mL of Whole blood per animal to be submitted in ONLY 10mL purple or green topped tubes. Sample tubes must be inverted repeatedly after collection and the use of older/dated tubes should be avoided to maintain the viability of the sample.

If you would like to submit a tissue sample for your animals please contact Pfizer Animal Genetics Customer Service.

After collection, the samples should be clearly labeled with the animal's identification details and refrigerated until they are able to be couriered to the office of Pfizer Animal Genetics, at the below address:

Pfizer Animal Genetics
Level 4
9 Moray Place
Dunedin 9016
Phone 03 477 5920

The samples need to be couriered in cold storage (an ice pack in a Styrofoam box will suffice) along with a signed sample submission form and a signed copy of the terms and conditions, both available from Customer Services (0800 228 278) or by download from:

http://www.pfizeranimalgenetics.co.nz/sites/PAG/nz/Pages/Testing_and_Results.aspx

Should you have any questions please don't hesitate to contact us on 0800 228 278

Kind Regards,

Pfizer Animal Genetics

Sheep50K

For more information please contact Pfizer Animal Genetics

P: 0800 228 278

P: 64 3 477 5920

F: 64 3 477 5930

E: pahgenetics.nz@pfizer.com

W: www.pfizeranimalgenetics.co.nz

This technology is a result of New Zealand
farmer investment in **Beef + Lamb**
New Zealand and **Ovita Ltd.**